

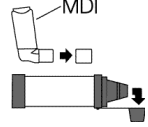
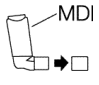
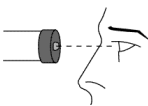
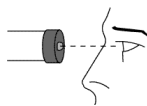
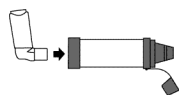
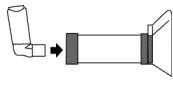
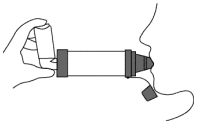
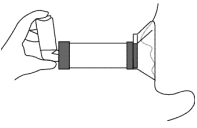
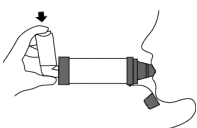
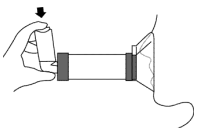
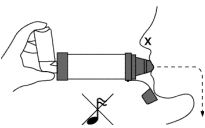
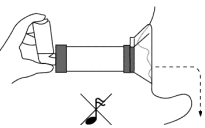
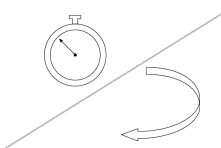
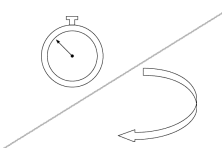


Directions for Use (Single Patient Use Only)

| | | |
|---|--|--|
|  |  | <p>Important: Follow instructions that came with inhaler (MDI).</p> <p>Note: Directions are for single patient use only.</p> |
|  |  | 1. Remove cap(s). |
|  |  | 2. Look for foreign objects. |
|  |  | 3. Insert inhaler (MDI) into back. |
|  |  | 4. Apply mask to face or put mouthpiece into mouth. |
|  |  | 5. Depress inhaler (MDI) at beginning of slow inhalation. If using mask, maintain good seal for 2–3 breaths after depressing inhaler. |
|  |  | 6. Slow down inhalation if you hear the FLOWSIGNAL [®] whistle sound. |
|  |  | 7. Wait 30 seconds before repeating steps 4–6 as prescribed. |

Notes

- VHC = Valved Holding Chamber
- Do not leave AeroChamber Plus VHC unattended with children. This is a medical device, not a toy.
- Do not disassemble the product at any time (even for cleaning, except for flexible back piece).
- This product contains no latex.

Cautions

- Product should be replaced if valves are damaged or missing.
- If unsure on how to use this product, talk to your physician or pharmacist.
- Administer one puff at a time.
- RX only.